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REF 02-1-001

DESCRIPTION

Constructed from breathable elastic fabric with flexible stays for dorsal (thoracic) support. Features adjustable hook-and-loop shoulder straps that cross at the back, with a posterior hook-and-loop (Velcro) closure. The shoulder straps are padded with foam for enhanced comfort. Color: Beige

INDICATIONS

- Early-stage kyphosis
- Gradual improvement of thoracic posture.

WARNINGS

- Do not use this product without prior advice from your healthcare professional or qualified therapist.
- Do not use the product if the size is incorrect. Selecting the correct size is essential for proper fit, optimal performance, and user safety.
- Always apply the product to clean and dry skin.
- Do not apply the product over areas with skin lesions, bleeding disorders, swelling, or varicose veins without consulting your physician or healthcare professional.
- Discontinue use if discomfort, skin irritation, or pain occurs.
- Do not use the product if it is deformed or damaged.
- Do not attempt to repair or modify the product yourself.

SAFETY INSTRUCTIONS

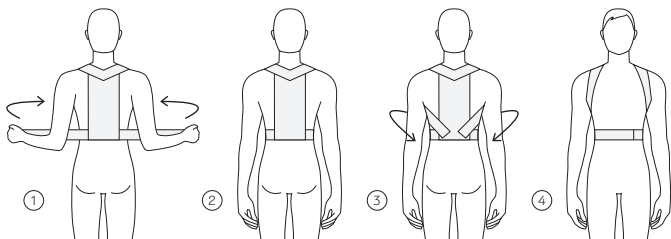
- Read these instructions carefully before using the product.
- Always follow the instructions for use to ensure safe and effective application.
- Inspect the product before each use for signs of wear, tears, or deformation. Do not use the product if any damage is detected.
- Incorrect use or failure to follow these instructions may result in complications or injury.
- This product is intended for use by a single patient only and should be used according to the instructions of a physician or physiotherapist.
- If pain occurs, persists, or worsens, discontinue use immediately and consult your physician or physiotherapist.
- The manufacturer is liable only for manufacturing defects, provided that the product has been used in accordance with these instructions.
- The manufacturer's liability and warranty apply only to products purchased through authorized distributors or specialized retailers.

CLEANING & MAINTENANCE

- Clean the product regularly, at least once a month or more frequently if necessary, to maintain it in optimal condition.
- Hand wash with lukewarm soapy water at a temperature of up to 30°C.
- Do not iron, bleach, dry clean, or tumble dry.
- Store the product in a clean, dry place and avoid exposure to extreme temperatures.

INSTRUCTIONS FOR USE

1. Position the wider section of the brace, containing the stays, against the back (posterior) of the torso.
2. Fasten the front waist strap using the hook-and-loop (Velcro) closure.
3. Pass the two hook-and-loop (Velcro) shoulder straps under the arms and secure them in the desired position.
4. Make sure the product fits properly.

ΜΕΓΕΘΗ / SIZES	SMALL	MEDIUM	LARGE	X-LARGE
Περίμετρος θώρακος Chest circumference	80-90 cm	90-100 cm	100-110 cm	110-120 cm
Ύψος ζακέτας Brace height	35 cm	39 cm	44 cm	49 cm

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